

## CADD Prizm Pump - Continuous Profile

### Set up and Starting the Infusion

Remove the battery from the pump, flip it over and put it back in the pump with the correct charges. (+ to + and - to -)




**\*\*\*\*\*Allow the pump to completely power up before attaching the IV tubing. You will hear 6 beeps to confirm the pump has successfully powered up.\*\*\*\*\***

### Spiking the Bag and Priming the Tubing

- If the bag and tubing are not already attached and primed, spike the IV tubing into the medication bag port. Squeeze the bag to prime the tubing to remove the air until you see a drip on the end of the tubing. Allow the filter to hang straight down. You do not need to flip or rotate the filter.
- Remove the blue clamp from the cassette once the tubing is primed.

### Attach the Cassette

- Attach the cassette hooks into the hinge pins on the pump. Place the pump on a hard surface and press the cassette down firmly.

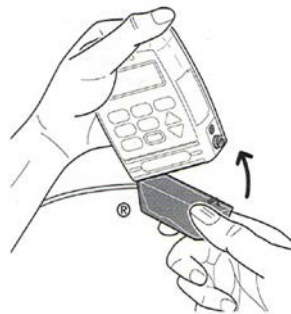

- Hold the pump securely. Using a coin, press in and turn the latch on the side of the pump counterclockwise until the mark on the latch is completely straight up.




- Screen will read “**High or low volume admin. set latched.**” High or low volume set will depend on the infusion rate.

HIGH OR LOW VOLUME  
ADmin set latched  
  
Next to continue

- Press “**NEXT**” to advance. Screen will read, “**Prime Tubing?**”. Press **N** (NO.).
- Screen will read, “**Start the Pump? Press Y or N.**” Press **Y** (Yes).
- The pump will read, “**Starting Pump**” and the pump will automatically flip through the program and then start the infusion. The green light on the top right hand side of the pump will begin flashing on and off.

### **Performing the Bag and Tubing Change**


- Press “**STOP**”. “**Stop the Pump?**” Press **Y** (Yes).
- Perform the bag and tubing change. Latch the cassette onto the bottom of the pump.
- Screen will read “**High or low volume admin. set latched.**” High or low volume set will depend on the infusion rate.
- Screen should read, “**STOPPED**”.
- Press “**NEXT**” to advance to the Reservoir Volume. The Reservoir Volume should be reset back to the amount of your bag/infusion.
- If the “Res Vol” has not automatically reset, perform these steps:
  - Press “**ENTER**” while on the Reservoir Volume screen.
  - Screen will read, “**Reset Reservoir Volume to XXXX ml?**”
  - Press **Y** (Yes). Screen will read “**Reservoir Volume XXXX ml Resetting. Infusion Profile has been reset.**”
- Screen will read, “**Prime Tubing?**” Press **N** (No). “**Start the Pump?**” Press **N** (No).
- Press “**NEXT**” to advance through the program until you get to the “**Milliliters Given**” screen. Press “**ENTER**” to clear the amount infused/given back to “**0**” (zero).
- Press Start. Screen will read, “**Start the Pump?**” Press **Y** (Yes).

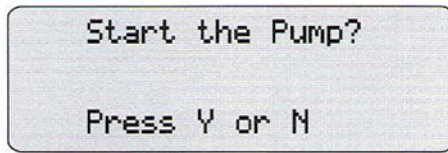

- Screen will read “**Starting Pump**” and then flip through each screen of the program.
- Screen will read “CONT” and the word “**RUNNING**”.
- The “Res Vol” on the screen should read the amount of your bag/infusion.

### **Stopping the Infusion/Turning the Pump Off**

- Press “**STOP**”. Screen will read, “**Stop the Pump?**” **Press Y or N.**

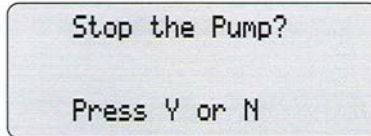

- Press “**Y (Yes)**”. Remove the bag and tubing from the pump and discard in the trash.
- Remove the battery from the pump, flip it over and put it back in the pump with the charges reversed. Lay the pump in a clean, safe place until you are ready to hook up the pump.